

# HAMA-JET™

## DAILY

مطهر و منظف أجهزة الشفط - يومي

# Daily Aspirator Cleaner and Disinfectant



1111-010

1111-009

**For the Daily cleaning and maintenance of Dental Aspirator Systems.  
Can be used on wet line, dry line and semi wet/dry line suction systems.**

### Key Product Features

Highly effective cleaning and disinfecting product. Also used for effective cleaning of the spittoon bowl. Non-foaming and non-abrasive, aldehyde, chlorinne and phenol free solution. Easy to use and measure. Colour coded. Effective against a wide spectrum of organisms. (see table opposite). Alkaline based for removal of Daily deposits.

### Directions for use

Dispense 5% solution: 50ml Daily concentrate to 950ml warm water. Aspirate via the high and low volume suction tubes and also dispense into the spittoon bowl. Leave for a minimum of 10 minutes in order to activate the working properties and then rinse thoroughly with clean water.

Time of action 10-60minutes

### Composition


Contains amongst other ingredients: Polyaminopropyl Biguanide, less than 5% nonionic surfactant, EDTA, Disinfectants.

### Physical Properties

Blue liquid, Odour characteristic, pH 10.0 - 11.0. Soluble in water, non-flammable with a shelf life of 3 years.

### Packaging

1111-010 1 Litre Container bottle  
1111-009 5 Litre Container  
Constrate 50 ml Equals 1 Litre  
Constrate 1 Litre Equals 20 Litre  
Constrate 5 Litre Equals 100 Litre

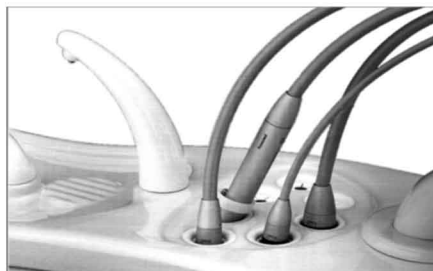

### Effective Spectrum

|   |                         |
|---|-------------------------|
| ✓ | <b>Bacteria</b>         |
| ● | MRSA                    |
| ● | E-Coli                  |
| ● | Pseudomonas aeruginosa  |
| ● | Enterococcus hirae      |
| ● | Staphylococcus Aureus   |
| ✓ | <b>Viruses</b>          |
| ● | HIV 1                   |
| ● | Human Influenza B Virus |
| ✓ | <b>Fungus</b>           |
| ● | Aspergillus niger       |
| ● | Candida albicans        |

