

HAMA-ULTRAZYME™

مطهر و منظم لتوصيلات المياه لوحدة الأسنان

Dental Water Line Cleaner



1111-024

A combination of both disinfectant and detergent for the routine decontamination of dental water lines.

Key Product Features

dental water line cleaner disinfectant concentrate Active Long lasting disinfectant effect Against Bacteria and Fungi Effective against algae and limedeposits and preventlimesacle formation when warming water or after water cool down Non-foaming Fresh Mint Flavor Biode gradable according to classification by the CE certificate european approval pH (undiluted) Approx. 6.0

Directions for use

Measure 20ml to 1,000 ml of warm water (30-40°C). Ultrazyme™ will mix with water instantly. Use the solution at the end of a clinical session. Switch off air supply, remove water bottle and empty of remaining water and refill with the diluted solution. Reconnect the water bottle and restore the air supply. Purge each individual line to refill with the solution, expelling any residual air. Allow to stand for a minimum of 10 minutes. Switch off air supply: remove the water bottle, discard any remaining contents and rinse the water bottle thoroughly, prior to refilling with clean water. Reconnect the bottle, restore air supply and purge each line to rinse thoroughly ensuring that any residual solution is replaced with fresh water.

مطهر و معقم لخطوط
المياه لوحدة الأسنان

Product Composition

Ensure that each line is rinsed for at least 45 seconds.

Physical Properties

Aqua, Isopropyl Alcohol, Propylene glycol, C9-11 Pareth-9, Subtilisin, Parfum

Packaging

Red in colour, Odour Characteristic.
1111-024 5 Liter Concentrate
1% (1 Liter Concentrate = 100 Liters)

