

HAMA- Oil Spray™

زيت تشحيم قبضات
وحدات الأسنان

Universal Handpiece Oil Lubricant Spray

All purpose Hand Piece Lubricant Spray for maintenance of all dental handpieces including high speed, slow speed and air motors.



1111-014

Key Product Features

Highly effective lubrication for all high and slow speed dental rotary handpieces. Complex oil with silicone and anti-rust inhibitors. Anti-rusting agent prevents micro rust on inner bearings. High penetrative ability to ensure thorough coating of bearing surface. Each can contains a high volume pressure to provide maximum bearing rotation during oiling. High internal pressure of 70+Psi (4.8atm) at 20°C. Cans are to UN specification and compatible with Kavo, W&H, Bien Air etc. Oil nozzles.

Directions for use

Insert into back of handpiece and spray through for one to two seconds immediately after each use and before sterilisation. Wipe excess thoroughly. Important: Hold can in a vertical position when in use. Refer to handpiece maintenance instructions where applicable.

Physical Properties

Clear in colour, Aerosol, Extremely flammable, peppermint aroma, shelf life of 3 years.

Packaging & Accessories

500ml Universal Oil Spray
Square section steel can complete with visco nozzle mounting facility.
For Slow & High Speed Nozzle.

Product Composition

White mineral oil, contains trace of peppermint oil.

